


▶ ALLIGATORK

The ergonomic torque device
operable with one hand


ALLIGATORK

- Press and release handling system
- 2 diameters:
between 0.010" and 0.020"
between 0.025" and 0.040"

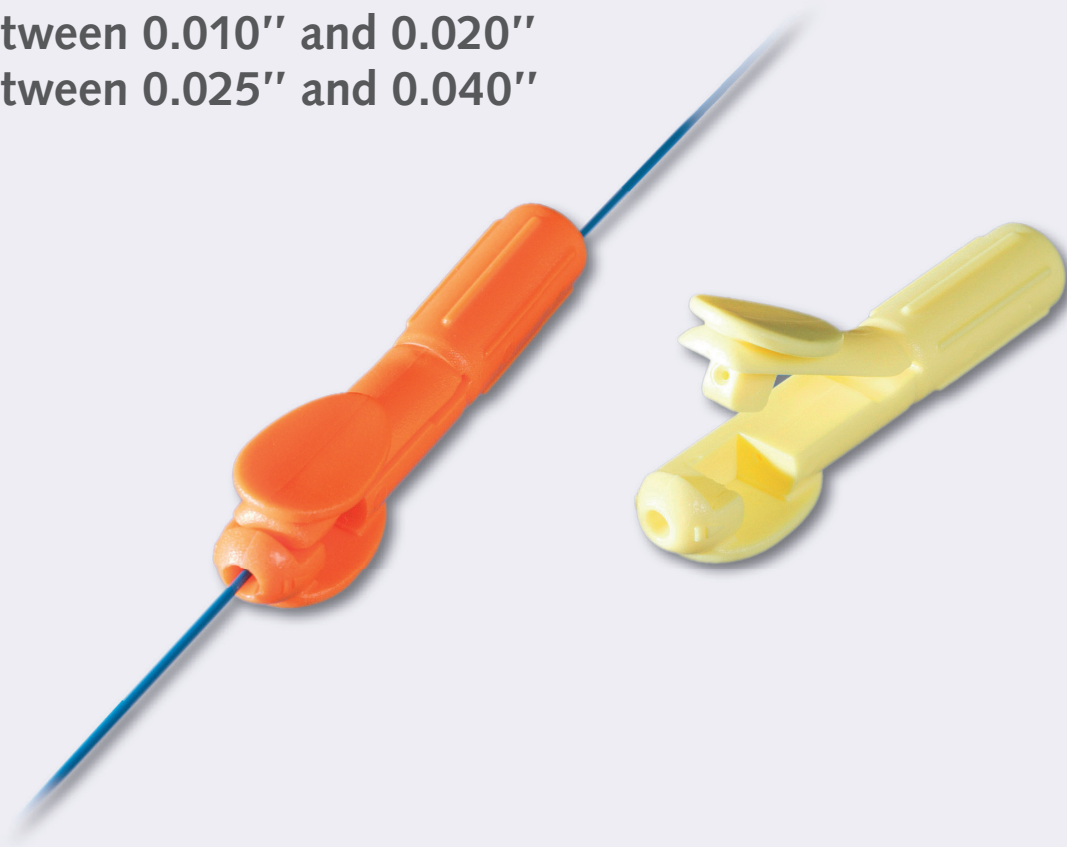





○ INSTRUCTIONS FOR USE

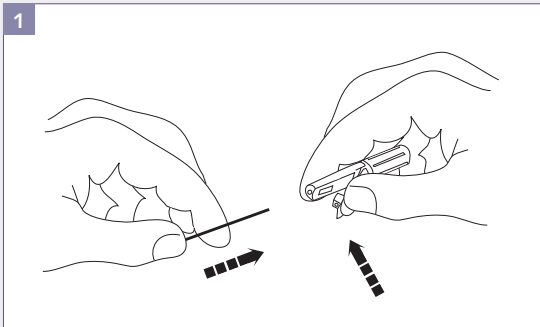


Pressing lever firmly and inserting guidewire

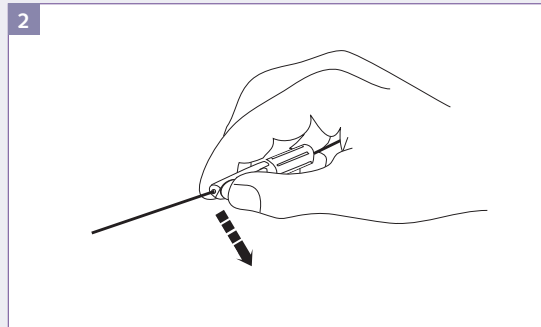

Positioning torque device and releasing lever

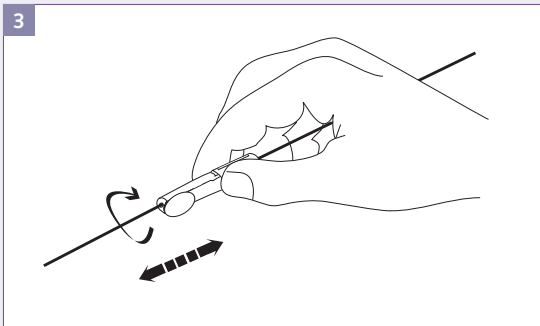

Positioning your guidewire

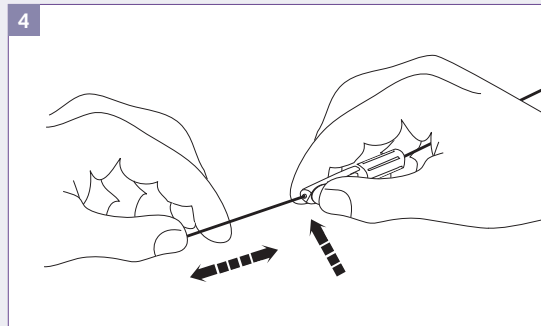

Pressing lever firmly to move or remove torque device

References	Description	Packaging
0203NA	Yellow for guidewires with diameter between 0.010" and 0.020" <i>Single packed and sterile (EtO)</i>	48-unit box
0203NC	Orange for guidewires with diameter between 0.025" and 0.040" <i>Single packed and sterile (EtO)</i>	48-unit box

Please contact PEROUSE MEDICAL for non sterile and OEM products.
This medical device is CE marked by LNE-GMED (0459).